

BS-1000A-1 FACIAL STEAMER WITH TIMER





1. Brief

1. Introduction of Function:

The facial steamer is used to improve blood circulation open the pores, and generally revitalize the skin. The addition of O3 to the steam helps to further soften the solidified sebaceous caps so that they can be more easily removed. A natural germicide, O3 enriched steam helps promote a more sterile environment.

2.Feature:

- 1. Thermal protector circuit shuts off steamer automatically if water gets too low.
- 2. Easy to clean water container and heating element.

5. Specification

Voltage / Hz	☐ 100V~120V ☐ 220V~240V
Wattage	650W
Ampere	6A
Size	50L*85W*125H(CM)
Weight	4.5kgs

4 . Caution

- 1.After water cup is cool, remove and wash with soap and water.
- 2. Heating element may be cleaned by changing the distilled water in container every time by changing the treatment. Clean residue from element using clear water and soft cloth, Use no metal or abrasives.
- 3. The water in the cup must be distilled water.
 No other product or liquid (such as aroma therapy oils) can be added to the distilled water supply.
- 4. The oils can be filled in the Essential Oil Reservoir.

NOTE: Some essential oils may be very corrosive and may damage the integrity of the plastic nozzle.

The use of corrosive essential oils is not recommended and cannot be covered under the One Year Limited Warranty.


2°. A s s e m b l y



3 . A s s e m b l y

Stand Assembling:

- 1. Put the black holder(3)on cross stand(4), then place the white tube on it. Lock screw(7) and washers (5&6) at the bottom of cross stand.
- 2. Please twist the adjust holder(1) before put on stainless tube(8). When assembling the length of the stand; be careful not to look directly down on it. The coil inside the stand could pop up unexpectedly and cause serious injury.

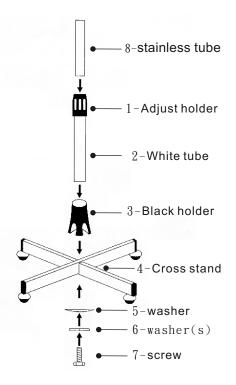

Operation:

- Fill the glass jar through the supply inlet with distilled or purified water.
- 2.Turn the steam power switch on. The red light indicate the heating element is operational.
 Allow 5 to 7 minutes for steam to start flowing from the steam nozzle.
- 3. Activate the O3 by turning on the O3 power switch.
 The green light indicates the O3 feature is functioning.
- 4. Based on customer's skin types, the following treatment times are recommended:

Normal Skin 8 to 10 minutes

Oily Skin with poor circulation 12 to 15 minutes

Sensitive or Dry Skin 3 to 5 minutes

Skin with inflammations or skin 3 to 5 minutes

with broken capillaries

5. As the water is depleted from the container, refill as needed through the inlet. The optimum operating capacity is at the red line on the container.