



100%
chiropractic.

www.100percentchiropractic.com

Tri-Core® Water Pillow

Adjustable Cervical Support

- Helps reduce headaches and neck pain for better sleep
- Oval shaped center in water chamber gently cradles head and neck
- Easily control support level with premarked water chamber
- Instantly responds to changes in sleeping position
- Customize level of support by adding or removing water



26" x 15" • White

**Core Products**
INTERNATIONAL, INC.
"Making YOUR Life More Comfortable."
Manufactured for 100% Chiropractic

CORE PRODUCTS®

TRI-CORE® WATER PILLOW

ADJUSTABLE CERVICAL SUPPORT

INSTRUCTIONS FOR USE

Provides support to the head and neck.

To fill water chamber: Step 1: Flip up tabs on cap and twist cap to remove. Step 2: Fill water chamber with distilled water. Step 3: Hold water chamber vertically and fill to desired level of support using guide at side of water chamber. Step 4: Screw cap securely to prevent leaking. Step 5: Use cloth or paper towel to absorb excess water around fill port. Step 6: Place water chamber in pillow pocket and zip close.

Place this pillow in a pillowcase.

If you have questions about the use of this product, consult your health care professional.

WASHING INSTRUCTIONS

PILLOW:


Remove water chamber.

Zip cover closed prior to washing.

Machine wash using warm water, mild detergent, no chlorine additive, gentle cycle.

Run through rinse cycle twice and spin feature at least twice to get as much moisture out as possible.

Tumble Dry, low heat. Add tennis or dryer balls in the dryer to help prevent clumping.

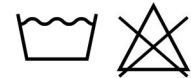

WATER CHAMBER:

Wipe exterior clean with damp cloth and air dry.

Rinse interior with mild disinfectant.

Rinse thoroughly and let air dry before refilling.

