


726 & R726

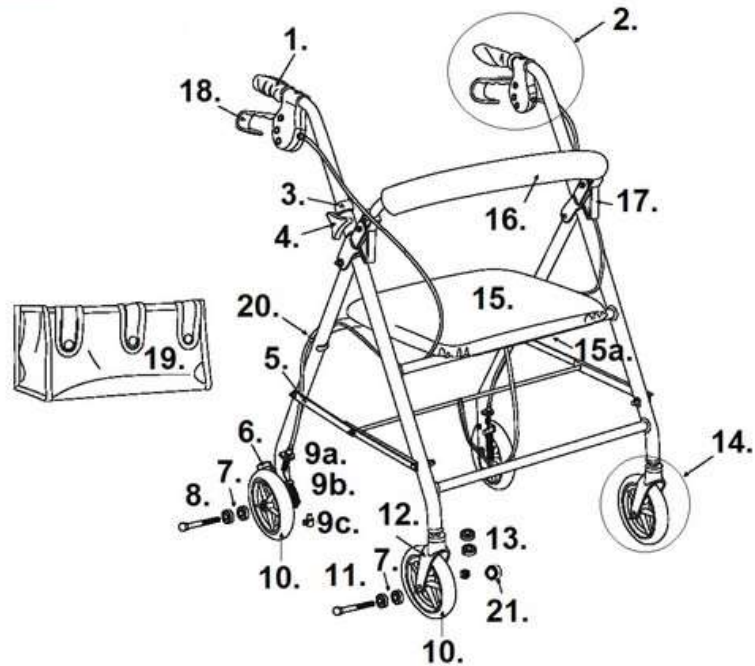

- | | |
|--|---|
| 1. Handgrip- 9501S72602 | 13. Fork Stem Bearing- STDS1060 |
| 2. Handlebar Assembly Left- 9501S72605L
Handlebar Assembly Right- 9501S72605R
Handlebar Assembly Left- 9520H72605L [3]
Handlebar Assembly Right- 9520H72605R [3]
(inc. Brake, Grip & Handle Bar) | 14. Fork-Caster Assy. 1S Serial #- [2]
Fork-Caster Assy. 2M, 9H & 20H Serial #- 9502M72601
Fork-Caster Assy.1P Serial #- 9501P72601 |
| 3. Anti-Rattler- 950300169901 | 15. Seat- 9501S72604
Seat- 9520H72604 [3] |
| 4. Height Adj. Knob- 10208KN | 15a. Seat Brackets 1S Serial #- 950340059901
Seat Brkts. 2M, 1P, 20H & 9H S/N- 950340059901-2M |
| 5. Folding Hinge Assembly- 9501S72609 | 16. Backrest Red- [1]
Backrest Blue- [1]
Backrest Black- 9501S726BK
[1] Stock is gone use 9501S726BK |
| 6. Wheel Lock Right- 9501S7261R
Wheel Lock Left- 9501S7261L | 17. Backrest Pivot Bracket- 9501S72603 |
| 7. Wheel Bearing- 9500002211 | 18. Hand Brake with Cable- 9501S72607
Hand Brake with Cable- 9520H72607L [3]
Hand Brake with Cable- 9520H72607R [3] |
| 8. Rear Axle- 9501A79506-2M | 19. Tote Bag- 9505W1026120 |
| 9a. Cable Adjuster- 9500001232 | 20. Frame Clip- 10208C |
| 9b. Spring- 9501S72616 | 21. Axle Cap- 950330149901 |
| 9c. Cable Fastener- 9500004101 | [2] These parts are no longer be available. |
| 10. Wheel- 9501S72600 | [3] 20H serial number |
| 11. Front Axle- STDS2A3423 | |
| 12. Fork Assembly 1S Serial #- [2]
Fork Assembly 2M, 9H & 20H Serial #- 9502M72610
Fork Assembly 1P Serial #- 9501P72610 | |