



NRG Digital Massage Table Warmer 231-0293

The NRG Digital Massage Table Warmer is designed to provide gentle warmth to your clients. The pad will not feel warm to the touch, but your clients will notice the heat when they lay on your dressed table.

CAUTION

- Do not use on swollen, irritated or broken skin. People with diabetes, poor circulation or neurological insensitivity should not use this pad.
- Discontinue if pain or other unusual symptoms appear. Consult your therapist for guidance if you are unsure.
- Do not use on children or adults who are unable to remove or monitor the pad.
- Do not use as a bed warmer or foot warmer.
- Do not place on surfaces sensitive to heat.

STORAGE

- Do not fold or roll tightly as this will damage the wiring and void your warranty.
- Operating your NRG Digital Massage Table Warmer when folded or bunched up could lead to product failure, injury, overheating or fire. Improper use or care will void your warranty.
- Unplug the warmer when not in use. After the warmer has cooled completely it may be loosely folded or rolled up for storage.

CLEANING


- Do not machine wash or dry. The product may be spot cleaned with warm water and mild soap and allow to air dry. Allow to dry completely before using.

WARRANTY

- One year limited warranty from date of purchase. Please read all instructions carefully to avoid the possibility of voiding this warranty.



CONTROLLER INSTRUCTIONS

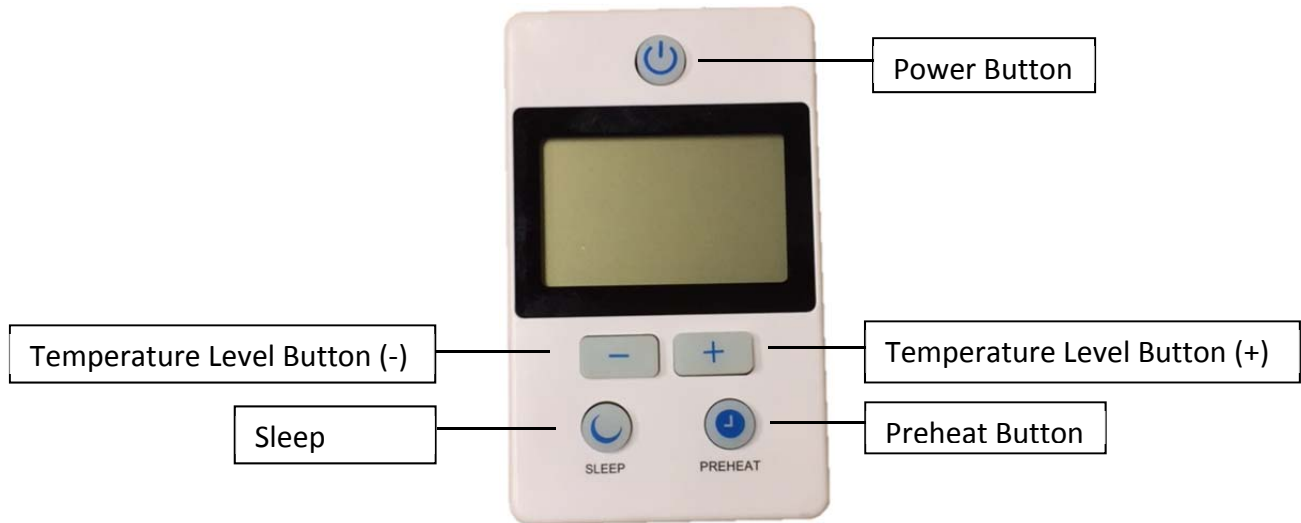

Setup and Usage

- Connect the plug from the controller to the warmer, and insert the polarized plug into an outlet. A buzzer will sound, the LCD will display light for 3 seconds, and then the warmer will turn off.
- Unfold the Digital Massage Table Warmer and lay it flat on your table.
- The warmer should be placed on top of your table, followed by fleece pad (if desired), fitted sheet, flat sheet and blanket. This premium table dressing keeps your clients cozy and comfortable while retaining the warmth of the pad. Table dressing should be adjusted to fit your climate, season and client preference.

Button and Display Functions

- Power Button: Push to turn on/off unit. The screen will display a number indicating the current timer setting and a temperature scale indicating the temperature level.
- Preheat Button: Press the preheat button to warm your pad. The heat setting will increase to Hi and the timer will be set for 60 minutes.
- Temperature Level Buttons: Use the +/- buttons to modify the temperature. Use minus (-) button to decrease temperature and plus (+) button to increase temperature.
- Sleep Button: Press sleep button to set the timer. Time range is from 10 minutes to 99 minutes.

Troubleshooting

If the controller makes a beeping sound and displays an "F" on the screen, immediately unplug it and check the connection. After ensuring the connection is secure and the correct voltage power receptacle is being used, plug the cord into the outlet and the warmer should function properly. If the controller continues to beep and display an "F" on the screen discontinue use and contact Massage Warehouse.